

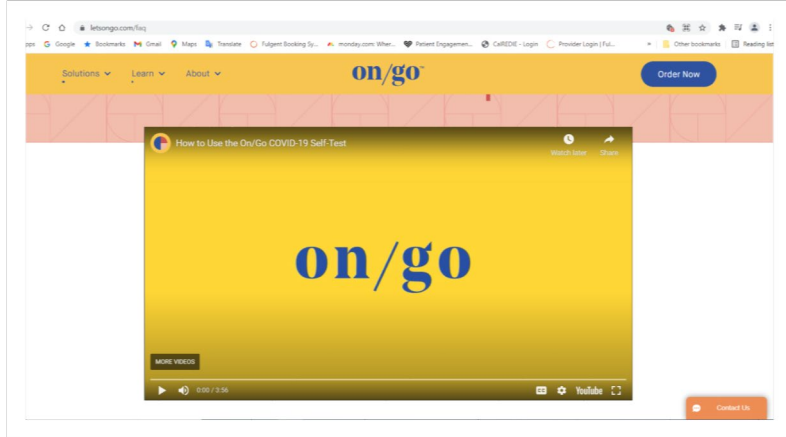


STEPS TO COMPLETE YOUR COVID-19 ON/GO ANTIGEN SELF-TEST


STEP 1:
Watch the On/Go instructional video at letsongo.com/faq

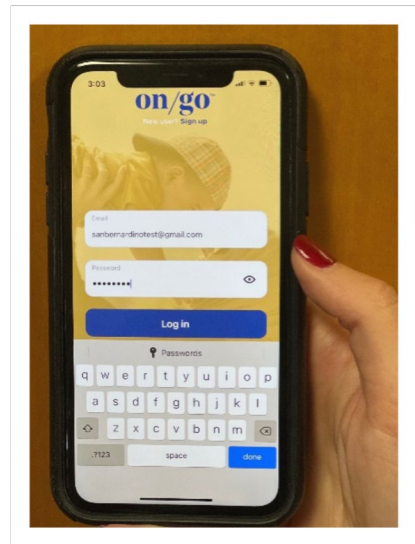


STEP 2:
Download the On/Go results app

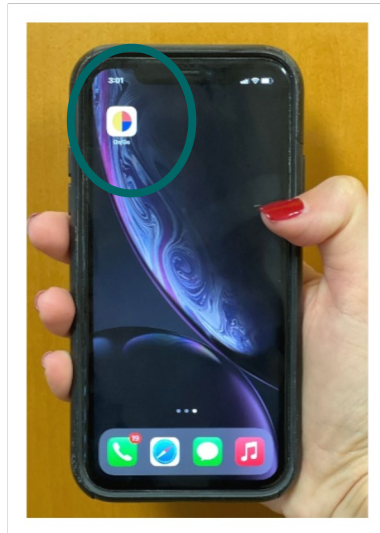


STEP 3:
Follow the instructions on the On/Go app

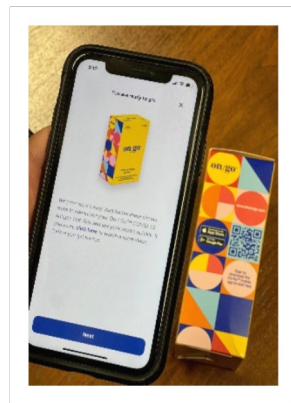






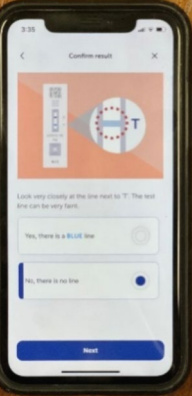
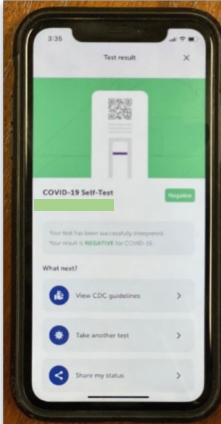

Scan the box QR code

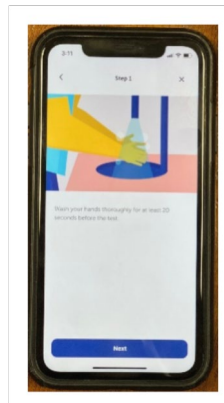

Wash your hands first

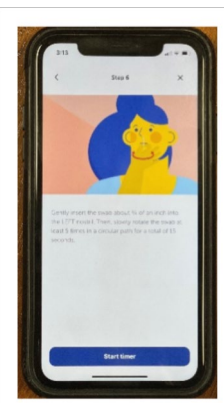

Swab front section of each nostril for at least 15 seconds

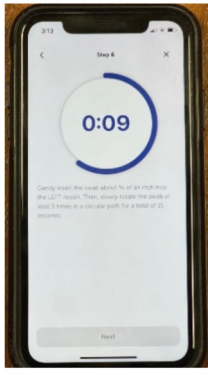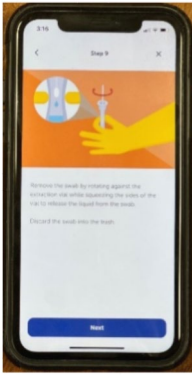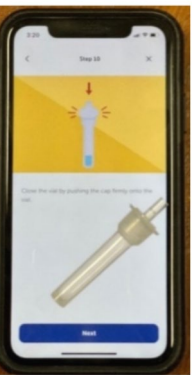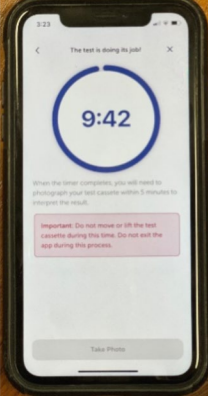			
	<p>The app will time you</p>	<p>Swab with mucus will be placed in solution</p>	<p>Lid will be snapped onto mixed solution</p>
			
	<p>The app will indicate a 10 minute time</p>	<p>After 10 minutes, you will have 5 minutes to take a picture of your results</p>	<p>The app will guide you to confirm the results</p>
<p>STEP 4: View results</p> <p>All results are saved on the On/Go app to be viewed anytime and can be shared at your discretion with a healthcare provider, employer, or other</p>			

NOTE: The On/Go antigen testing app contains a total of 12 steps to receive your results. The steps shown above are quick reference and not intended to be comprehensive.